

PerCP anti-human CD3


Analyte Specific Reagent. Analytical and performance characteristics are not established.

Catalog# / Size	981016 / 500 μL
Clone	SK7
Other Names	Τ3, CD3ε
lsotype	Mouse IgG1, κ
Description	CD3 ϵ is a 20 kD chain of the CD3/T-cell receptor (TCR) complex, which is composed of two CD3 ϵ , one CD3 γ , one CD3 δ , one CD3 ζ (CD247), and a T-cell receptor (α/β or γ/δ) heterodimer. It is found on all mature T cells, NK T cells, and some thymocytes. CD3, also known as T3, is a member of the immunoglobulin superfamily that plays a role in antigen recognition, signal transduction, and T cell activation.

Product Details

Reactivity	Human					
Formulation	Phosphate-buffered solution, pH 7.2, containing 0.09% sodium azide and 0.2% (w/v) BSA (origin USA).					
Preparation	The antibody was purified by affinity chromatography, and conjugated with PerCP under optimal conditions.					
Concentration	200 μg/mL					
Storage & Handling	The antibody solution should be stored undiluted between 2°C and 8°C and protected from prolonged exposure to light. Do not freeze .					
Application	Suggested for Flow Cytometry					
Disclaimer	WARNINGS AND PRECAUTIONS					
	 Use appropriate personal protective equipment and safety practices per universal precautions when working with this reagent. Refer to the reagent safety data sheet. This antibody contains sodium azide. Follow federal, state and local regulations to dispose of this reagent. Sodium azide build-up in metal wastepipes may lead to explosive conditions; if disposing of reagent down wastepipes, flush with water after disposal. All specimens, samples and any material coming in contact with them should be considered potentially infectious and should be disposed of with proper precautions and in accordance with federal, state and local regulations. Do not use this reagent beyond the expiration date stated on the label. Do not use this reagent as these may be an indication of possible deterioration. Avoid prolonged exposure of the reagent or stained cells to light. 					

Product Data

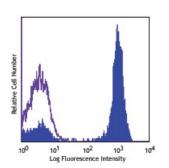

Typical results from human peripheral blood lymphocytes stained either with SK7 PerCP used at 5 μ L/test (filled histogram) or with an isotype control (open histogram).

Symbols Glossary*

Symbol	Meaning	Symbol Title	Symbol No.	Symbol	Meaning	Symbol Title	Symbol No.
REF	Catalog number	Catalogue number	5.1.6	Ĩ	Indicates the need for the user to consult the instructions for use.	Consult instructions for use	5.4.3
X	Indicates the temperature limits to which the medical device can be safely exposed.	Temperature limit	5.3.7	紊	Indicates a medical device that needs protection from light sources.	Keep away from sunlight	5.3.2
K	Indicates the upper limit of temperature to which the medical device can be safely exposed.	temperature	5.3.6	Σ	Indicates the date after which the medical device is not to be used.	Use-by date	5.1.4
••••	Indicates the medical device manufacturer.	Manufacturer	5.1.1	EC REP	Indicates the authorized representative in the European Community.	Authorized representative in the European Community	5.1.2
LOT	Indicates the manufacturer's batch code so that the batch or lot can be identified.	Batch code	5.1.5	IVD	Indicates a medical device that is intended to be used as an in vitro diagnostic medical device.	In vitro diagnostic medical device	5.5.1

be supplied – Part 1: General requirements

Not for resale. BioLegend will not be held responsible for patent infringement or other violations that may occur with the use of our products.

*These products may be covered by one or more Limited Use Label Licenses (see the BioLegend Catalog or our website, www.biolegend.com/ordering#license). BioLegend products may not be transferred to third parties, resold, modified for resale, or used to manufacture commercial products, reverse engineer functionally similar materials, or to provide a service to third parties without written approval of BioLegend. By use of these products you accept the terms and conditions of all applicable Limited Use Label Licenses.

BioLegend Inc., 8999 BioLegend Way, San Diego, CA 92121 www.biolegend.com Toll-Free Phone: 1-877-Bio-Legend (246-5343) Phone: (858) 768-5800 Fax: (877) 455-9587